REVITIVE®

Essential

Drug Free/Sans Médicament
Clinically Proven/Efficacité Cliniquement Prouvée
Certified Medical Device/Dispositif Médical

EN **User's Manual**

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Please read the user's manual carefully before using this product.

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USE ONLY AS DIRECTED. If symptoms persist, consult your healthcare professional

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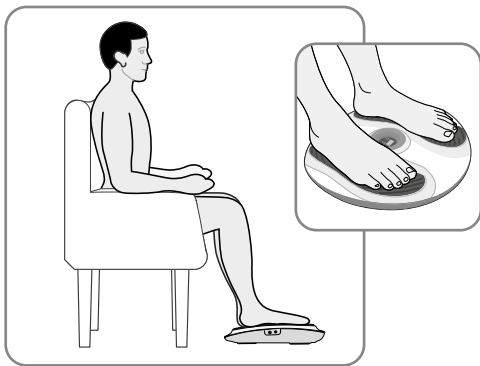
 Intensity **40+**
 **30-60** min

 for **60+** days

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For videos of setup and using your Revitive system, go to: support.revitive.com

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
Table of contents

- What is inside the box?** _____ **4**
- Parts and controls** _____ **5**
- Introduction to Revitive** _____ **6**
 - Operator profile _____ 6
 - How Revitive works _____ 6
 - Indications for use _____ 6
- Important safeguards** _____ **7-10**
 - Who should **NOT** use Revitive _____ 7
 - Warnings and cautions _____ 8-9
 - Safety precautions _____ 10
 - Adverse reactions _____ 10
- Instructions for use** _____ **11-18**
 - How do I get the most from Revitive? _____ 11
 - Step 1: Setting up Revitive for the first time _____ 11
 - Step 2: Using the foot-pads _____ 12-13
 - Step 3: Using the small electrode pads _____ 14-17
 - Cleaning and storing _____ 18
- Replacing the small electrode pads** _____ **18**
- Troubleshooting** _____ **19**
- Technical specifications** _____ **20-22**
- Guidance and manufacturer's declaration** _____ **23-24**
- Warranty** _____ **25**


What's in the box?

Upon opening the carton, please check that the following components are provided. If you think anything is missing, please contact us using the helpline numbers on the back of this booklet.

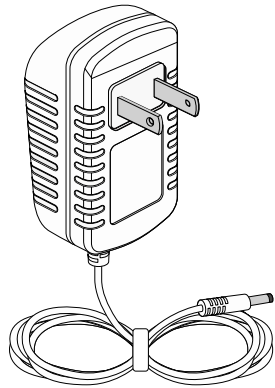- 1** Power adaptor
Cable length 180cm


- 2** Small electrode pads (2044)

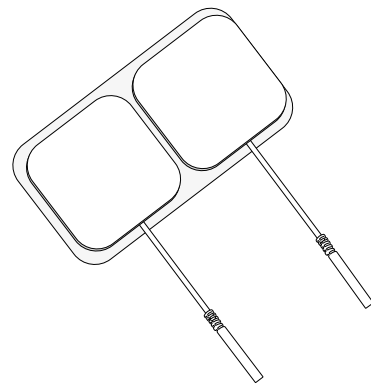- 3** Revitive Essential



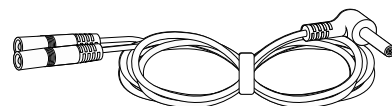
- 4** Electrode cord
Cable length 150cm

Parts and controls

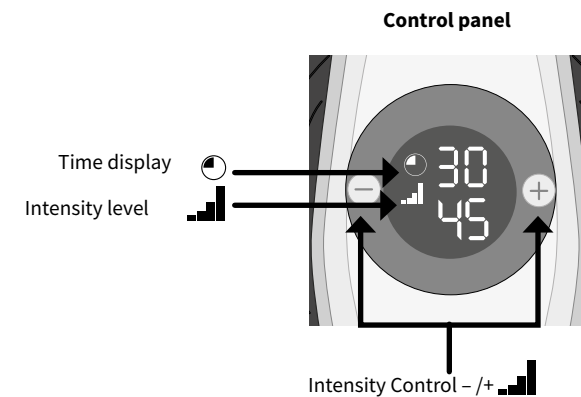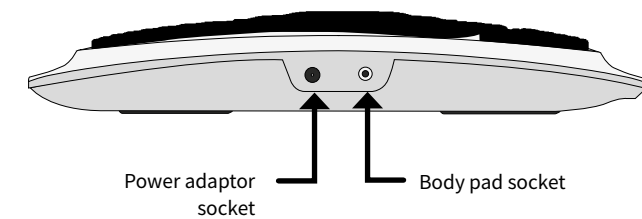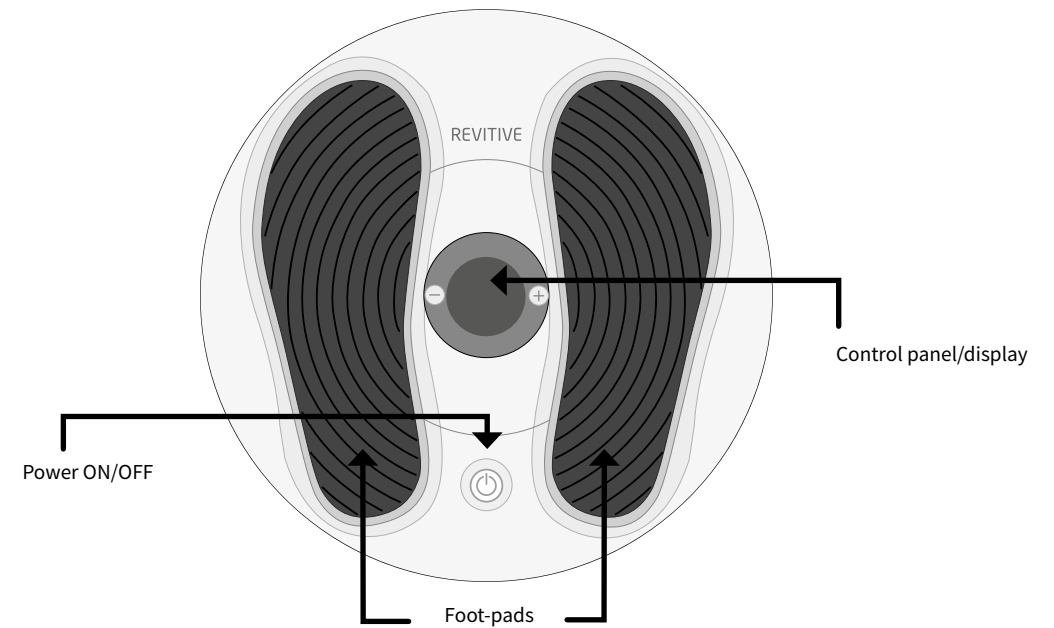EN

Operator profile

The intended operator profile is based on users in the home environment who may have a sedentary lifestyle or experience long periods of sitting or standing and related issues such as pain, leg stiffness and swelling in the legs, ankles and feet.

Revitive is intended for use by the end user in a non-clinical setting and without the supervision or intervention of a clinician during use.

A typical user may be of any adult age range and is not specific to any demographic of gender, ethnicity or educational background.

How Revitive works

Revitive delivers Electrical Muscle Stimulation (EMS) to your feet via a pair of cushioned foot-pads, while you are seated. The electrical stimulation causes muscles in your calf and foot to contract, which helps to increase circulation in your legs.

Revitive has been developed so that it is comfortable for you to use.

Revitive can also deliver targeted electrical stimulation through the use of the electrode pads provided

Indications for use

Revitive is intended to, when using the foot-pads:

- Improve circulation to reduce or prevent blood-pooling (stasis) - caused by poor circulation due to a sedentary lifestyle or long periods of sitting or standing
- Help maintain leg vein health – by increasing circulation, delivering more oxygenated-blood in the legs

The above Indications for Use are certified under the Medical Devices Directive 93/42/EEC.

In addition, if you have a sedentary lifestyle or spend long periods inactive, Revitive may help to:

- Strengthen leg muscles
- Reduce foot & leg discomfort
- Actively increase circulation.

Visit www.revitive.com for more information

Read all instructions before use.

Save these instructions.

Revitive should not be used by some people.

Do not use if (contraindications):

- You are fitted with an electronic implanted device such as a heart pacemaker or Automatic Implantable Cardioverter Defibrillator (AICD)
- You are pregnant
- You are being treated for, or have the symptoms of, an existing Deep Vein Thrombosis (“DVT”)

Long periods of inactivity can put you at greater risk of developing Deep Vein Thrombosis (DVT). DVT is a blood clot and usually occurs in a deep leg vein. If part of the DVT breaks off it may lead to potentially life-threatening complications such as pulmonary embolism.

If you have been inactive for prolonged periods and suspect you may have a DVT, consult your doctor immediately.

To prevent dislodging the clot do not use Revitive.

In some cases of DVT there may be no symptoms. However it is important to be aware of the symptoms that may include:

- Pain, swelling and tenderness in one of your legs (usually your calf)
- A heavy ache in the affected area
- Warm skin in the area of the clot
- Redness of your skin, particularly at the back of your leg, below the knee

Consult your doctor as soon as possible if you show any signs of the above symptoms.

Warnings

Consult with your doctor before using this device if:

- You are in the care of a doctor
- You have a history of heart problems
- You have had medical or physical treatment for your pain
- You have suspected or diagnosed heart disease
- You have suspected or diagnosed epilepsy
- You are unsure about the suitability of the device for you
- You are unsure about the cause of your symptoms

If you have a metallic implant:

- If you have a metallic implant, you may experience pain or discomfort near the implant when applying electrical stimulation. If this should occur discontinue use and seek advice from your doctor
- The electrical stimulation may feel more intense close to a metallic implant. It is safe to continue use provided no pain is experienced. You may need to adjust the intensity to a comfortable level

When applying foot-pads or electrode pads:

- Do **not** use the electrode pads on your head, face, neck or chest
- Application of the electrodes near the thorax (chest) may increase the risk of cardiac fibrillation
- Only apply to areas with intact/unbroken skin
- Do not apply foot-pads or electrode pads directly or adjacent to these areas:
 - on cut or broken skin, open wounds or rashes; swollen, red, infected or inflamed areas; or skin eruptions (such as phlebitis, thrombophlebitis, varicose veins, cellulitis). Check under your feet and toes before using the device
 - on, or close to, malignant tumours
 - on areas treated with radiotherapy (within the past 6 months)
 - on reproductive organs
 - on opposing thighs and refer to pad placement diagrams in this User's Manual for correct placement
 - over the carotid sinus nerves (front of the neck), particularly in patients with a known sensitivity to the carotid sinus reflex
- Electrode pads are for single person use only. Do **not** share electrode pads with others
- Do **not** use electrode pads after their use-by date
- There is a chance you may experience skin irritation or burns beneath the electrodes or hypersensitivity due to the electrical stimulation
- Ensure that any water based moisturiser is evenly applied and thoroughly absorbed into the skin before applying stimulation. There is a chance that uneven application could increase the risk of skin irritation or burn, when using the electrical stimulation
- Symptoms may worsen during the initial treatment phase before getting better. This may occur if the body has not fully adjusted to increased muscular activity and blood circulation. If this occurs, reduce the intensity and the duration of treatment which will reduce the initial symptoms. If symptoms persist consult with your doctor

When not to use Revitive:

- Do not use the device:
 - In the presence of electronic monitoring equipment
 - Together with a life-supporting medical electronic device
 - When you are in the bath or shower
 - While you are sleeping
 - In conjunction with a brace or cast without first consulting with your doctor

Warnings

When not to use Revitive:

- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally

Additional warnings:

- Use of accessories and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Revitive System, including cables specified by the manufacturer

Cautions

When applying foot-pads or electrode pads:

- Be careful when applying stimulation over areas of skin that lack normal sensation. It may cause skin irritation due to the inability to feel stimulation until the intensity is too high. Use a low intensity to achieve a gentle muscle contraction, and/or use for a shorter time, to avoid over-stimulation. Check for signs of skin irritation (redness), bruising or pain. If in doubt consult your doctor
- This product is not intended for use by persons with reduced physical, sensory or mental capabilities, unless they are supervised by a person responsible for their safety

When using EMS:

- Be careful when applying stimulation:
 - Over the abdomen during menstruation
 - After recent surgical procedures (within the past 6 months) as stimulation may disrupt your healing process
 - If your tissues are likely to bleed following an injury such as a muscle tear - it is recommended not to stimulate the immediate area within the first 12 hours after sustaining the injury. Use a low intensity and/or shorter time to avoid over-stimulation
 - After a long period of immobility or inactivity – use a low intensity and shorter time to avoid over-stimulation or muscle fatigue

Additional cautions:

- The long term effects of electrical stimulation are unknown
- Electrical stimulation may not work for every user, please seek advice from your doctor

Safety precautions

- Do not stand on the machine. Use only when seated
- Do not position the Revitive so that it is difficult to disconnect or turn off
- Use the device only for its intended purpose
- Do not expose the device to extreme heat
- Do not spill liquid on the device or its accessories
- Do not overload the electrical socket
- Keep device out of the reach of children
- Keep power cords and cables out of the reach of children to prevent risk of strangulation
- Revitive foot-pads may be used by multiple persons. Ensure device is cleaned after each use
- During use, do not touch foot-pads with your hands until the paused feature is enabled or device has powered off
- Use Revitive only with the accessories supplied by, or purchased from, the manufacturer
- Check electrode pads, cords and cables periodically for damage
- Do not open Revitive or repair it yourself. This will invalidate your warranty and may cause serious harm
- In the unlikely event your Revitive malfunctions, disconnect it from the power source and contact your nearest authorised agent
- It has been reported that some universal remote control devices (e.g. for TV etc.) can change the settings on Revitive if used during a treatment. If this should occur, simply adjust the time or intensity settings on Revitive back to where you want them using the Revitive Remote Control, or using the device Control Panel
- Revitive has passed the required tests for Electromagnetic Interference (EMI); it may still be affected by excessive emissions and/or may interfere with more sensitive equipment
- After any exposure to hot or cold temperatures outside the specified operating range of 10 – 40°C allow the product to re-adjust to the recommended operating temperatures to ensure continued product performance

Adverse reactions

- If you experience adverse reactions, stop using Revitive and consult with your doctor immediately

How do I get the most from Revitive?

For best results:

- Use **Revitive foot-pads for at least 30 to 60 minutes each day**, 7 days a week.
- It is important to use Revitive at a high enough intensity to give you a strong comfortable muscle contraction. Most people achieve a strong muscle contraction **over intensity level 40** (intensity range 1-50).
- If you are diagnosed with a long term medical condition such as High Blood Pressure, High Cholesterol, Diabetes, Osteoarthritis, Chronic Venous Insufficiency (CVI), Peripheral Arterial Disease (PAD), or COPD, it can take **up to 8 weeks** to help reduce symptoms associated with these chronic conditions.

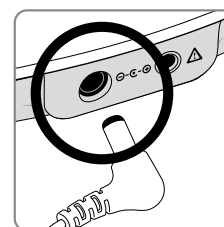
Do not use Revitive, using EMS stimulation, for more than 6 sessions of 30 minutes (or the equivalent) per day.

Step 1: Setting up Revitive for the first time


a Remove all parts from the box:

- AC/DC power adaptor
- Revitive Essential device
- Small electrode pads
- Electrode cord

See “What’s in the box?” on page 4 to help identify the parts.

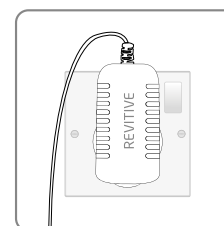- #### b
- Locate the power adaptor.
Plug the small end of the power cord into the socket on the side of the Revitive.



- #### c
- Plug the power adaptor into the nearest electrical socket.

Before you use Revitive for the first time, read the Important Safeguards on pages 7-10. If in doubt, consult your doctor before using the product.

Step 2: Using the foot-pads

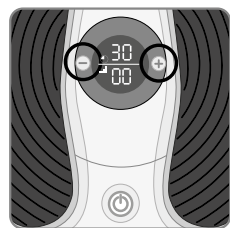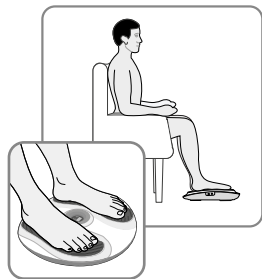Sit with both bare feet on the foot-pads

- a** It is important that you are well hydrated. We recommend that you apply a moisturiser to the soles of the feet to help hydrate the skin and improve the delivery of electrical stimulation. Please use a water based moisturiser such as the Revitive Foot and Leg Cream.
- We also recommend drinking a glass of water before using Revitive.
- b** Get into a comfortable seated position. For best results sit with your knees at a 90 degree angle (right angle).
- c** Place Revitive on the floor in front of you.
- Revitive is designed to be used while seated.
Never stand on the Revitive unit.
- d** Place **both bare** feet on the foot-pads.
Make sure you remove all footwear, including socks/stockings.
- e** Press the power button on the device or on the remote control to turn Revitive on.
- You will hear a beep and the display will light up. The ⌚ time will read 30 minutes and 📶 intensity will read 00.
- To turn off Revitive at any time during operation, press the power button on the device or on the remote control.
- f** Press the 📶 (+) Intensity Control one time to start stimulation.
- Press the (+) button to increase the level.
Press the (-) button to decrease the level.
- g** Ensure you familiarize yourself with the feeling of Electrical Muscle Stimulation (EMS). Use on a low intensity for 30-60 minutes once per day for a few days.
- Set the intensity at a level where you can feel the stimulation in your feet and which creates a calf contraction.
- It is normal to feel tingling or varying sensations in your calf muscles.
- Revitive does **not** vibrate.

- h** After a few days of familiarization, use a higher intensity that provides you with **strong but comfortable calf muscle contractions**.
- Most people experience a strong contraction at intensity level 40 or over. You can set the intensity between 1-50.
- Please note that the intensity required may vary from day to day.

If you feel pain or discomfort:

- Remove one or both feet from the foot-pads to stop the stimulation.
- Lower the intensity of the foot-pads using the 📶 (-) Intensity level button on the control panel before replacing your feet on Revitive.

- i** The ⌚ Time Display will start to count down in minutes as Revitive cycles through its program.
- j** When Revitive times out, the ⌚ Time Display reads 00 and you will hear three beeps.
- k** The Revitive switches itself off automatically.

Step 3: Using the small electrode pads

 Electrode pads are for single person use only. Do not share electrode pads with others.

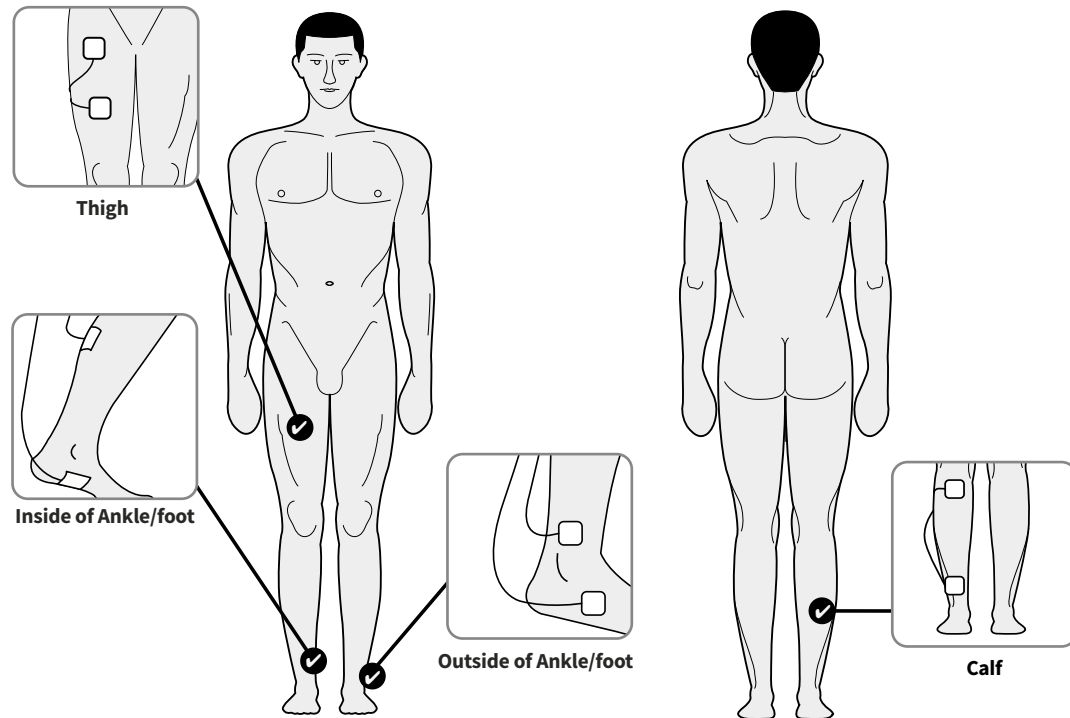
Revitive comes with one pair of reusable (up to 20-30 applications) electrode pads that can be used to deliver targeted electrical stimulation.

**The electrode pads cannot be used at the same time as the foot-pads.
Using the electrode pads will automatically turn the foot-pads off.**


Use Revitive electrode pads in EMS mode as and when required, from 30 minutes per day.
Do not use Revitive in EMS mode for more than 6 sessions of 30 minutes (or the equivalent) per day. This may result in muscle fatigue.

a Select the area you wish to apply electrical stimulation to.

 Example small electrode pad placements are shown below:

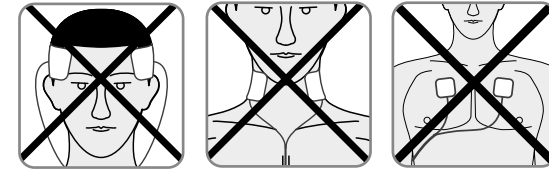
Do **NOT** use the electrode pads on your head, face, neck or chest.

b Clean and dry the area where you will place the electrode pads.

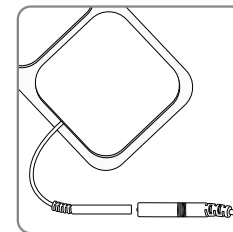
c Make sure Revitive is switched off. If the control panel is still illuminated, press the power button and turn Revitive off.

d Attach the ends of the electrode pad to the electrode cord.


Press connectors until there is no gap between them.

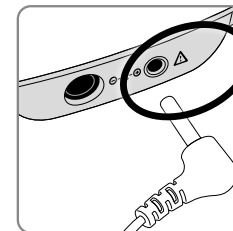Leaving a gap may cause discomfort or no stimulation.


e Plug the other end of the electrode cord into the socket on the side of Revitive.

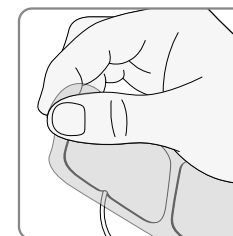Plugging in the electrode cord will automatically turn the foot-pads off.

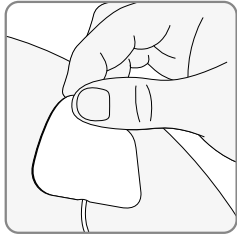
f Remove the plastic liner from the electrode pads, by peeling it off carefully. Replace the electrode pads onto the plastic liner after use.

- g** Press the electrode pads on the area you wish to apply electrical stimulation to, as shown on page 14, gently pressing the adhesive side against the skin.

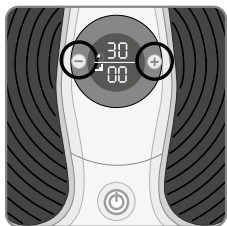- h** Press the power button on the Revitive.
You will hear a beep and the display lights will read 30 minutes.
To turn off the device at any time during operation, press the power button.

- i** Press the (+) Intensity Control one time to start stimulation.
Press the (+) button to increase the level.
Press the (-) button to decrease the level.

- j** Ensure you familiarise yourself with the feeling of Electrical Muscle Stimulation (EMS). Use on a low intensity for 30 minutes once per day for a few days.

It is normal to feel tingling or varying sensations in your muscles you are treating.

- k** After a few days of familiarisation, use a higher intensity that provides you with **strong but comfortable muscle contractions**.

You can set the intensity between 1-50.

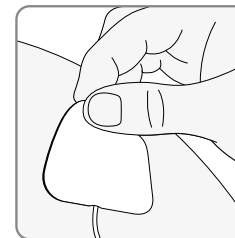
Please note that the intensity required may vary from day to day.

- l** The Time display will start to count down in minutes as Revitive cycles through its program.

- m** When Revitive times out, the Time display reads 00 and you will hear three beeps, and the device will switch itself off automatically.


If you feel pain or discomfort:

- Lower the intensity of the electrode pads using the (-) Intensity Control.
- You can also choose to stop therapy before the end of the 30 minute session to avoid skin irritation. You can do this by pressing the power button.


- n** Ensuring that the device is switched off, gently peel the electrode pads from the skin.

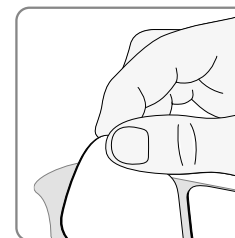Do not pull the electrode pads off the skin using the electrode cords as this may damage the cords.

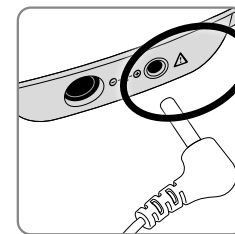- o** Replace the electrode pads onto their plastic liner after use.

- p** Unplug the electrode cord from the socket on Revitive.
You do not need to detach the electrode pads from the electrode cord.

Cleaning and storing Revitive


- a** Ensure Revitive is switched off.
- b** Wipe down the foot-pads with a soft damp cloth.
Do not clean with chemicals.
Do not immerse Revitive in water.
- c** Store Revitive in a cool, dry and dust-free location.
Store out of direct sunlight.
- d** Place electrode pads onto the plastic liner.
- e** Store the electrode pads in a dry and ventilated location.
Store out of direct sunlight.
You do not have to detach the electrode pads from the electrode cords to store them.

Replacing the small electrode pads (2033)

The electrode pads are reusable up to 20-30 applications. If the electrode pads start to lose their adhesiveness, rub a few drops of water into the surface. Replacement electrode pads can be obtained via www.revitive.com.

Problem	Possible cause	Solution
Revitive is on (lights illuminated on the LED display) but I cannot feel the electrical stimulation through the foot-pads.	Not placing both bare feet on the foot-pads at the same time.	Ensure that your feet are bare and each foot is placed on each of the foot-pads at the same time. Keep increasing the intensity up to a maximum of 99 until you feel the stimulation.
	Your feet may be dry.	Moisturise the soles of your feet to improve conductivity and stimulation and try the procedure again. You may also have to increase the intensity level.
	You may be dehydrated.	Drink plenty of water before and after using the device. The device uses your body to create the electrical circuit. Water is an excellent conductor of electricity and if your body is less hydrated (below 60%) then the stimulation may be less, therefore it is important to always remain well hydrated.
	The intensity level may be on too low a setting.	This is a very safe device. Keep increasing the intensity level towards 99 until you feel the stimulation. You may find that you have to increase the intensity level as you get used to the therapy. The aim is not to get to 99 but to find a setting that produces strong muscle contractions in your calves and is comfortable for you.
	Electrode cord is connected to device.	Disconnect the Electrode cord - the foot-pads will not work while the Electrode cord is plugged in.
	If, having tried the solutions above, you still cannot feel the stimulation:	Test the device by placing one hand across both foot-pads at the same time (The heel end of the foot-pads is easiest). With your other hand, and starting from zero, increase the intensity level until you can feel the stimulation. If you can feel the stimulation through your hand then the device is working. If on 99 you still cannot feel the stimulation then please contact your authorised dealer.
No power or lights to Revitive when it is switched on.	AC Adaptor not switched on at the electrical socket or AC Adaptor not plugged into device properly.	Check electrical socket power is switched on and the AC Adaptor is plugged into the device correctly. If still not working – contact your authorised dealer.
Revitive is not vibrating.	REVITIVE IS NOT DESIGNED TO VIBRATE.	
My legs are aching after treatment.	You may have the intensity on too high a setting and your muscles are being overworked.	Leave adequate time after each treatment to allow the muscles to recover (just like after vigorous exercise!). On your next session start on a lower setting (where you can feel the mild electrical stimulation and it is comfortable) and reduce the duration until your muscles have acclimatized to the stimulation.

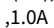For more information including Frequently Asked Questions, please visit www.revitive.com

Name of product	Revitive Essential
Model	RLV
Weight (+/-0.5kg)	1.13 Kg
Dimensions	Ø 355 mm x 77(D) mm
Power consumption	5W
AC adaptor	CE Approved
Power source Input (adaptor used) ⚡ Output ⚡	100-240V ~ AC (⊕), 50/60Hz, 0.18A 5V (⊕) DC  , 1.0A
Service Life of Device Period of time specified by the manufacturer during which the device is expected to remain safe for use.	4 Years
Applied parts Part of Revitive that in normal use come into physical contact with user to perform therapy	Foot-pads - 135.5cm ² Small electrode pads - 5cm x 5cm = 25cm ²

Output Specifications for Electrical Muscle Stimulation (EMS):

Waveform	Biphasic		
Shape	Square symmetrical with polarity reversal		
Frequency (+/-10%)	Foot: 20 to 44Hz	Body: 37 to 51Hz	
Max Charge / pulse at level 50	@500Ω	Foot: 0.02mC	Body: 0.02mC
Max Average Current at level 25	@500Ω	Foot: 6.80mA (RMS)	Body: 6.0mA (RMS)
Max Average Current at level 50	@500Ω	Foot: 9.7mA (RMS)	Body: 10.3mA (RMS)
Maximum Output Voltage at level 25	@500Ω	Foot: 13.5Vp	Body: 12.5Vp
Maximum Output Voltage at level 50	@500Ω	Foot: 21.5Vp	Body: 21.5Vp
Pulse Duration (+/-10%)	Foot: 450 or 970μs (Mode 7)	Body: 450μs	
Net Charge	@500Ω	[0.001]mC	
Maximum Power Density	@ 500Ω	Foot: 0.19 mW/cm ²	Body: 1.5 mW/cm ²
ON Time (+/- 10%)	Foot: 1.9 - 8.3s	Body: 1.9 - 6.5s	
OFF Time (+/- 10%):	Foot: 1.0 - 1.5s	Body: 1.0s	

Voltage and current waveforms


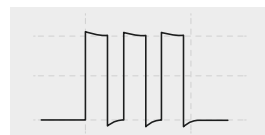
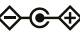
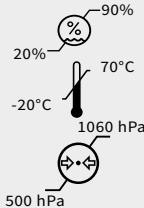
Image is representative of both foot and body waveforms and is the same at both levels 50 and 25.

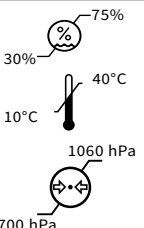The values of Pulse durations, Pulse repetition frequencies and amplitudes, including any D.C. component, do not deviate by more than ± 20 % when measured with a load resistance within the range specified.

If confirmation is required that the Revitive works within its Essential Performance after a certain period of time, contact the manufacturer.

Symbols

Complies with European Medical Devices Directive (93/42/EEC)	
UK Conformity Assessed Product conforms to all applicable U.K. legislative requirements.	
Device serial number including year (YYYY) and month (MM) of manufacture can be found on the box and back of unit	 #YYYYMMXXXX
Item number	
Model number	
Indicates the unique device identification reference	
Contraindications, Warnings and Cautions Make sure you understand these before using Revitive	
Power	
Time Remaining	
Intensity Level	
Center Positive Polarity	
Indicates the device is a Medical Device	
Class II medical electrical equipment double insulated	
Type BF medical electrical equipment	
Legal manufacturer of the device	
EU/EC European Authorised Representative	
Country distributor	
Made in South Korea	

FCC mark Certification mark employed on electronic products manufactured or sold in the United States which certifies that the electromagnetic interference from Revitive is under limits approved by the Federal Communications Commission. Actegy Ltd complies with all applicable FCC rules.	
Consult instructions for use	
The Waste Electrical and Electronic Equipment Directive (WEEE Directive). At the end of the product lifecycle, do not throw this product into normal household garbage, but take it to a collection point for the recycling of electronic equipment	
Ingress Protection Rating	IP21
Humidity, temperature and air pressure limit for storage and transport	
Humidity, temperature and air pressure limit for operating conditions After any exposure to hot or cold temperatures outside the specified operating range of 10 - 40°C allow the product to re-adjust to the recommended operating temperatures to ensure continued product performance.	
Indoor Use Only	
Medical device does not contain natural rubber latex	
Do not disassemble	
Medical — Applied Current/Energy Equipment As to electrical shock, fire and mechanical hazards only in accordance with standards: ANSI/AAMI ES 60601-1:2005/A2:2021 CAN/CSA-C22.2 No. 60601-1 (Amendment 2:2022) IEC 60601-1-6, Edition 3.2, Issue Date 07/2020 IEC 60601-1-11, Edition 2.1, Issue Date 07/2020 IEC 60601-2-10, Edition 2.2, Issue Date 01/2023	 CLASSIFIED C UL US E529786

Symbols - applies to electrode pads only

Use-by date	 YYYY MM DD
Made in China	
Single User, multiple use, medical device	

Important information regarding Electromagnetic Compatibility (EMC)

With the increased number of electronic devices such as PC's and mobile (cellular) telephones, medical devices may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the medical device and could create a potentially unsafe situation.

In order to regulate the requirements for EMC, with the aim to prevent unsafe product situations, the IEC/EN 60601-1-2 and IEC/EN 60601-1-11 standards have been implemented. These standards define the levels of immunity to electromagnetic interference as well as the maximum levels of electromagnetic emissions for medical devices.

Revitive devices are intended for use in the home setting without the need for involvement of a clinical professional during treatment. On occasion the device may also be used in a hospital environment. In this environment use near the following equipment should be avoided:

- active High Frequency Surgical Equipment
- radio frequency shielded rooms for magnetic resonance imaging (where the intensity of Electro Magnetic Disturbances is high).

Warnings

- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories and cables other than those specified or provided by Actegy Ltd for this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Wireless communications equipment such as wireless network devices, mobile phones, cordless telephones and their base stations can affect this equipment and should be used no closer than 30 cm (12 inches) to any part of the Revitive Circulation Booster, including cables specified by Actegy Ltd. Otherwise, degradation of the performance of this equipment could result.
- No modification of the Revitive device or equipment is allowed
- Connection to high frequency surgical equipment whilst using Revitive may result in electrode burns and possible damage to the device. Operation in close proximity (e.g. 1m) to shortwave or microwave therapy may produce instability in the device output.

FCC Caution

This device complies with Part 15 of FCC Rules.

Operation is subject to the following conditions:

- (1) The device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

IC Statement

This Class B digital apparatus complies with Canadian ICES-003.

Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Supplier's Declaration of Conformity 47 CFR § 2.1077 Compliance Information

Unique Identifier:	RLV
Company Name:	Actegy Health Ltd.
Address:	The Lightbox, Bracknell, RG12 8FB, United Kingdom
Telephone:	Tel: 800-732-6962
Contact name:	Sarabpreet Singh (regulatory@actegy.com)

Electromagnetic Compatibility Information

Phenomenon	Basic EMC standard or test method	Test level/requirement
Mains terminal disturbance voltage	CISPR11 EN 55011	Group 1, Class B
Radiated disturbance	CISPR11 EN 55011	Group 1, Class B
Harmonic Current Emission	IEC 61000-3-2 EN 61000-3-2	Class A
Voltage change, voltage fluctuations and Flicker Emission	IEC 61000-3-3 EN 61000-3-3	Pst: 1 Plt: 0.65 Tmax: 0.5 Dmax: 4% Dc: 3.3%
Electrostatic Discharge Immunity	IEC 61000-4-2 EN 61000-4-2	±8 kV/Contact ±2, ±4, ±8, ±15 kV/Air
Radiated RF Electromagnetic Field Immunity	IEC 61000-4-3 EN 61000-4-3	10V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz
Immunity to Proximity Fields from RF wireless Communications Equipment	IEC 61000-4-3 EN 61000-4-3	Table 9 in IEC 60601-1-2:2014
Electrical Fast Transient/Burst Immunity	IEC 61000-4-4 EN 61000-4-4	±2kV, 100 kHz Repartition frequency
Surge Immunity	IEC 61000-4-5 EN 61000-4-5	Line to Line ±0.5kV, ±1kV
Immunity to Conducted Disturbances induced by RF fields	IEC 61000-4-6 EN 61000-4-6	3V 0.15 MHz - 80MHz 6V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 KHz
Power Frequency Magnetic Field Immunity	IEC 61000-4-8 EN 61000-4-8	30 A/m 50 Hz & 60 Hz
Voltage dips	IEC 61000-4-11 EN 61000-4-11	0% U_T : 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T : 1 cycle and 70% U_T : 25/30 cycles Single phase: at 0°
Voltage Interruptions	IEC 61000-4-11 EN 61000-4-11	0% U_T : 250/300 cycle

It is important to retain the retailer's receipt as proof of purchase. Staple your receipt to this back cover for future reference.

Please quote the following information if the product develops a fault. These numbers can be found on the base of the product:

Model no:

Serial no:

All Revitive devices are individually tested before leaving the factory. In the unlikely event of any device proving to be faulty within 30 days of purchase, it should be returned to the place of purchase for it to be replaced.

If the fault develops after 30 days and within 24 months of original purchase, you should contact your local distributor quoting model number and LOT number on the product, or write to your local distributor at the address shown.

You will be asked to return the product (in secure, adequate packaging) to the address shown with a copy of proof of purchase.

Subject to the exclusions set out below (see Exclusions) the faulty device will then be repaired or replaced and dispatched, usually within 14 working days of receipt.

If, for any reason, this item is replaced during the 2-year guarantee period, the guarantee on the new item will be calculated from the original purchase date. Therefore, it is vital to retain your original till receipt or invoice to indicate the date of initial purchase.

To qualify for the 2-year guarantee, the device must have been used according to the manufacturer's instructions supplied.

To activate your free 2-year warranty please register your device at: revitive.com/ca/warranty

Exclusions

- Actegy, manufacturer of Revitive devices, shall not be liable to replace the goods under the terms of the guarantee where:
 - The fault has been caused or is attributable to accidental use, misuse, negligent use or used contrary to the manufacturer's recommendations or where the fault has been caused by power surges or damage caused in transit.
 - The device has been used on a voltage supply other than that stated on the product or used with a power adaptor other than the one supplied with the product.
 - Repairs have been attempted by persons other than our service staff (or authorised dealer).
 - The device has been used for hire purposes or non-domestic use.
 - The device is second hand.
- Actegy are not liable to carry out any type of servicing work, under the guarantee.
- Accessories such as electrode pads and bags are not covered by the guarantee.
- Remote control batteries and any damage from leakage are not covered by the guarantee.
- This guarantee does not confer any rights other than those expressly set out above and does not cover any claims for consequential loss or damage. This guarantee is offered as an additional benefit and does not affect your statutory rights as a consumer.

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Copyright © 2025 Actegy Ltd. All rights reserved. Actegy®, Revitive® and Circulation Booster® are registered trademarks of Actegy Ltd. The Revitive product is a proprietary design and is protected by applicable design laws
Copyright © 2025 Actegy Ltd. Tous droits réservés. Actegy®, Revitive® et Circulation Booster® sont des marques déposées d'Actegy Ltd. Ce produit Revitive est un design exclusif protégé par la législation applicable sur les dessins et modèles industriels.